Z chassis

Ⓛ	DE	Bedienerhandbuch Z Untergestell	5
Ⓛ	GB	User manual for the Z chassis.....	8

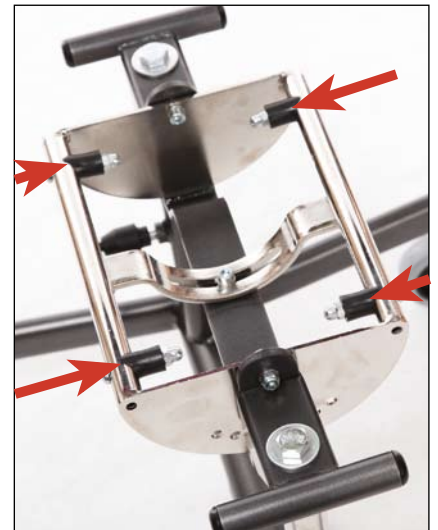
Z chassis + ThevoSeat

Z chassis + EASyS

(1)

(2)

(3)

(4)

(5)

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Dear user,

in order to make the operation of the Z chassis safe, practical, and comfortable for you and especially for your child, please read this user manual completely before use.

Should you have any further questions or issues, please contact your specialist supplier or Thomashilfen directly.

For more information about Thomashilfen and our products, please visit our website at: www.thomashilfen.com

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Contents

	Page
Safety-related checks and maintenance intervals.....	9
General safety instructions.....	9
Purpose and use.....	10
Risks and contraindications.....	10
Technical data	10
Installing the seat unit size 1	10
Installing the seat unit size 2	10
Seat height adjustment.....	10
Seat tilt-in-space adjustment.....	10
Parking brake function	10
Cleaning, disinfecting, and care instructions.....	11
Terms of guarantee / CE mark	11

Safety-related checks and maintenance intervals

- All operating elements and fastening screws should be checked every month. The telescopic sliding plate should remain free of dirt and sand.
- Please bear in mind that the screws and release handles (hip angle adjustment) should be secured after each adjustment.

Please observe the following general safety instructions:

- When handling Z chassis, especially when setting up, please observe the recommendations in the user manual.
- Please keep packaging material away from children to avoid the risk of choking.
- We recommend that you adjust the seat depth and lower leg length according to the current body size of your child at least every 3 months. If necessary, ask your therapist / orthopaedist for assistance.
- Please bear in mind to engage the locking device after adjusting the seat height, seat angle, and hip angle.
- Please be careful not to catch your fingers when carrying out adjustments.
- Be careful not to harm yourself when folding up the footrest or adjusting the seat height.
- **IMPORTANT:** Use the attached Allen key for all adjustments (Allen screw 3mm/0.1").
- If you have doubts about the correct use of Z chassis consult your local dealer to avoid incorrect adjustments.
- For safety reasons accessories not mentioned in this user manual should be mounted by specialists only.
- Please do not overload your Z chassis and observe the maximum load capacity (see technical data).
- Only one child should be seated in Z chassis.
- Do not leave your child unattended when getting in or off Z chassis. Excessive load of the footrest may lead to slipping off or tipping.
- We recommend that you always secure your child with the 2-point pelvic belt or another retention belt (available as an accessory), and never leave the child unattended in Z chassis.
- Always lock the brake before placing or removing the child from Z chassis.
- Always lock the brake when the chair is stationary to avoid rolling.
- Avoid any obstacles in the way of the chair, do not push the chair into them.
- **ATTENTION:** Wet wheels may affect the braking action. When stopping or halting for a longer period of time you should apply the foot brake to prevent inadvertent rolling.
- The Z chassis is supplied with the seat adapter mounted facing forwards. Never pull it out completely and replace it facing the opposite way. **RISK OF TIPPING OVER!**
- Stop your child from leaning out to the front or side. **RISK OF TIPPING OVER!**
- When lifting Z chassis make sure to hold it at tightly welded or tightly screwed component parts only (frame tube at front and rear wheels).
- Please avoid uneven ground, as this may cause Z chassis to tip over.
- Ground beams should be passed over only when the base frame is tilted to stand on the rear wheels (going upwards – pull backwards, going downwards – lowering slowly).
- Stairs must only be taken with the aid of an accompanying person. If available, elevators or similar equipment should be used. If such equipment is unavailable, the obstacle should be passed with the chair being carried by two persons, without the child sitting in it.
- Please be careful when pushing on slopes to avoid Z chassis tipping over or rolling away, or your child falling out of the chair.
- Parking Z chassis on slopes should be avoided.
- From time to time you should wipe off the dirt from the wheels. Use a damp cloth and make sure you dry thoroughly. Do not use strong solvent.
- High solar radiation may lead to high surface temperature on the dark upholstery material.
- Do not expose the gas springs to high pressure or temperatures.
- Do not loosen or remove any screws unless otherwise stated.
- Keep Z chassis away from naked flames.

Tip: You may find additional information at our web site www.thomashilfen.com

Purpose and use

The chassis is, in association with a compatible / convenient seat-system for ergonomic transfer of disabled children, suitable for indoor use.

Risks and contraindications

Proper handling may avoid any use-related risks. No contraindications are known.

Technical data

Seat tilt-in-space	-15° to +35°
Seat height with seat unit	39 - 58.5 cm / 15.4 – 23"
Total dimensions (l x w x h)	65 x 54 x 33 - 51 cm 25.6 x 21.3 x 13 - 20.1"
Size of swivel wheels (front)	3"
Size of swivel wheels (rear with brake)	3"
Weight (without seat unit)	7.2 kg / 15.9 lb
Carrying capacity	40 kg / 88.1 lb

Installing the seat unit size 1

When installing the seat unit, make sure that the longer section of the chassis arm is at the back (fig. 1). The seat unit must be positioned centrally between the spacers. (fig. 2)

Installing the seat unit size 2

Before installing the seat unit 2 on the Z chassis, the four spacers on the seat adapter must be removed. (fig. 2)

Seat height adjustment

CAUTION: Only adjust the seat height when the child is not in the seat. Adjusting with the child in the seat may cause the seat unit to tip unexpectedly with a risk of injury to the child.

Hold onto the seat unit firmly with one hand. Loosen the levers on each side. Now adjust the seat to the desired height and re-tighten the screws. (fig. 3)

CAUTION: Do not pull out the telescoping tube past the markings.

Seat tilt adjustment

Release the clamping lever beneath the centre of the seat unit, pull out the locking pin on the right of the seat adapter and adjust the seat to the desired angle. Then re-tighten the clamping lever. (fig. 4-5)

CAUTION: When adjusting the seat tilt, hold on to the back of the seat firmly in order to avoid sudden changes to the child's position. We recommend that you make these adjustments when the child is not in the seat.

Brake function

The rear castors are equipped with easily accessible and easy to use brakes. The brakes prevent the wheels both from turning and from swivelling, so that the Z chassis can be safely parked via the rear wheels alone.

Cleaning, disinfecting and care instructions

The Z chassis is manufactured from high-quality steel and aluminium tubing which is coated with a robust heat-enamel finish; and can be cleaned using commercially available no-toxic soap (neutral soap).

Terms of guarantee / CE mark

Thomashilfen offers you a guarantee of 3 years for all frame parts, commencing on the date of purchase. This guarantee incorporates all claims which influence functionality. Damages due to improper use (e.g. overloading), and natural wear and tear are excluded.

Z chassis meets the requirements of the European standards EN 12182 and 12183 and is provided with a CE marking.

Enjoy your Z chassis!

Member of the International
Support Association for the
Rehabilitation of Children and Youths e.V.

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Thomas Hilfen für Körperbehinderte GmbH & Co. Medico KG