ThevoTherapy

(DE)

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Therapietisch, klappbar

(GB)

Assembly instructions - Accessory

Tray, fold-away

(FR)

Notice de montage - Accessoires

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(ES)

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First, fix the tray receiver to the **right** armrest. Remove the screw for the angle adjustment of the armrest completely and fix the receiver tube with the screw and washer to the metal attachment under the armrest upholstery (fig. 1-2). Press the catch spring at the receiver tube of the tray and insert the tube into the receiver. Adjust the desired height with the knurled screw at the receiver (fig. 3). The angle and height adjustment is done by adjustment of the armrests.

IMPORTANT: The tray needs to bear completely on the armrests. Choose the cut-out side as close as possible to the child.

According to the chosen seat width the tray needs to be adjusted centrally. Loosen the four screws underneath the tray plate, adjust the corresponding position and retighten the screws (fig. 4).

The tray can be swung away to the side when not needed. When doing so pay attention not to hit the child with the tray. When swung away, screw the knurled screw tight to lock the tray (fig. 3).

CAUTION: The maximum load capacity of the tray is 2 kg / 4.4 lb. Never leave your child unattended in ThevoTherapy, especially when the tray is mounted.

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