A Untergestell Advantage / A Chassis Advantage

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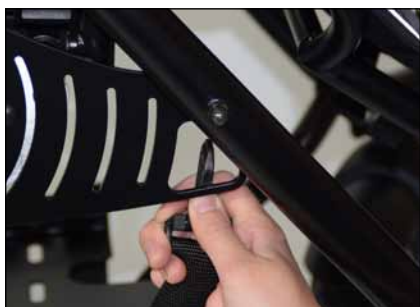
size 1 Item Code: 6294/1

size 2 Item Code: 6494/1

Insert the vent tray into the A chassis Advantage from the rear (see fig. 1). The clips on the vent tray should snap onto the rear axle (see fig. 2). Our recommendation is to snap the clips one after another. Please mind that there is a risk of fingers getting caught between clips and axle. Use the snap hooks at the front of the vent tray to attach it to the large openings at the front end of the side parts of A chassis Advantage (see fig. 3). The vent tray must be removed for folding the chassis.

When attaching the vent tray to an A chassis Advantage size 1, please mind the following instructions: After having mounted the vent tray like described above, two extra clips should be used (included in delivery). Insert the pins of the clips in the holes in the rear upper horizontal tube (see fig. 4). The clips protect the tube from possible scratches caused by the footrest when using the face-to-face position.

There are slots in the vent tray, which can be used for straps to secure medical devices (see fig. 5). Straps are not included in the scope of supply. Please keep in mind, that the maximum weight capacity of the vent tray is 5 kg (11 lbs).

Our tip: For easier loading, the footrest can be folded upwards (see fig. 6).

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Thomas Hilfen für Körperbehinderte GmbH & Co. Medico KG · Walkmühlenstr. 1 · D - 27432 Bremervörde · Germany