DE

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Short Guide: Tip Assist for A Chassis

Item Code: 6750

The tip assist is mounted at the right side of the rear frame (see picture 1). Please place the A Chassis on its side carefully on the floor, the right wheel should point upwards. Please ensure attention that the floor is clean to avoid damage to the paintwork.

Please loosen the plastic cap at the right rear wheel with a sharp item (e.g. screwdriver) to access the wheel bolt (see picture 2). Please loosen the wheel bolt with a ratchet handle (see picture 3). Pay attention that you do not scratch the rims. Take off the wheel and the wheel bolt from the rear frame of the chassis.

In the next step please place the tip assist on the rear frame so that the thru-holes are aligned (see picture 4). Afterwards install the rear wheel to the frame with the wheel bolt. Please tighten the bolt securely, place the plastic cap on top of the bolt and right the A chassis. Now the tip assist is ready for use.

!!! Important !!!

Please make sure that the rear wheel is fixed tightly and securely before you use the A Chassis again.