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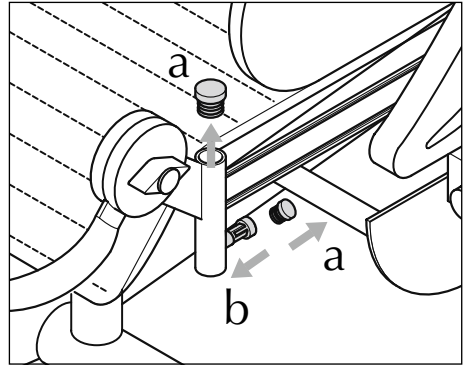
Mounting advice

Please mount the tray on a soft and clean surface to avoid that the tray plate gets scratched.

Care instructions

Clean the tray with a soft cloth to avoid scratches. When storing the tray, make sure that it cannot fall down to avoid scratches and cracks in the material.

1. Preparing the reception tubes

The reception tubes must be prepared before the tray is mounted: Remove the two plastic caps on the top and the side (a), and screw in the locking pins (b).

2. Mounting the framework

Mount the framework to the seat unit by putting the L shaped metal tubes into the reception tubes. Please make sure that the welded metal plates are situated on the **outside** of the L shaped metal tube. The L shaped metal tubes are adjustable in four (**Swiftly**: three) different heights by using the locking pins on the rear of the reception tubes.

3. Mounting the cross brace

Now mount the cross brace between the L shaped metal tubes. The holes are on the front of the metal plate (a). Afterwards cover the nuts with the plastic caps. Now, put a plastic disc onto the rear metal plate, both left and right. Make sure the corrugated side points outwards (**Swiftly**: inwards) (b).

4. Mounting the tray

Now turn the tray upside down and fix the metal brackets into the grooves on the left and right side using two screws and two washers on each side. Make sure that the metal plates face **outwards**. Afterwards put the plastic disk onto the metal plate on both sides. Make sure that the corrugated side points **inwards** (**Swiftly**: **outwards**).

5. Mounting the tray to the framework

Now you can mount the tray to the framework. Bolt the plastic disks together on the left and the right with the ruffles facing towards each other. Put the screw from the inside through both plastic disks and screw them tight with the washer and the nut. Now your tray is correctly mounted.

Adjusting possibilities:

- You can adjust the height of the tray using the locking pins at the reception tubes. You can choose between four (**Swiftly**: three) different heights.
- The angle of the tray is adjusted via the plastic disks. By slightly loosening the nut you can tilt the tray in 15° steps.
- To adjust the depth of the tray slightly loosen the four screws underneath the tray and slide it forwards or rearwards.
- If you like you can put pictures or mirror foil between the two tray plates. To do so, remove both white binder screws on the rear part of the tray plate. Now lift up the upper plate carefully (risk of breakage) by max. 3 cm / 1.2" and slide in the pictures or mirror foil. Now fix the plates with the screws again.

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Thomas Hilfen für Körperbehinderte GmbH & Co. Medico KG · Walkmühlenstr. 1 · D - 27432 Bremervörde · Germany