EASyS Advantage · Basic (EASyS Advantage S)

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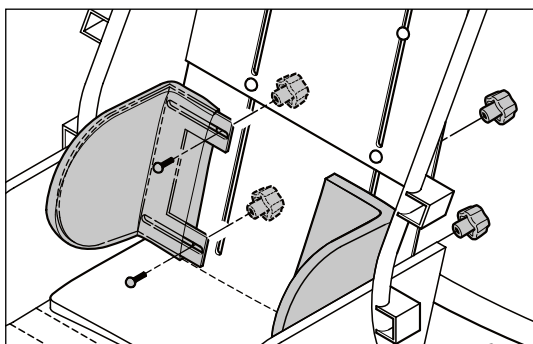
GB

Lateral trunk supports, stiff
size 1, Item code: 6634

size 2, Item code: 6734

In the back padding you will find two long, vertical slots. The Lateral trunk supports are led through these slots to fasten them to the backrest. The padding of the lateral trunk supports has to remain on the supports while mounting them on the backrest. The loose end of the padding at the inner side of the lateral trunk supports can be lifted for easier mounting. (fig. 1)

The lateral trunk supports must be mounted with the more sloping side pointing upwards. Please lead the lateral trunk support through the slot in the back padding. The outer part of the back padding should lie under the lateral trunk support while the middle part of the back padding should cover the support. (fig. 2)

Insert one screw in the upper slot of the lateral trunk support and the preferred vertical slot in the back rest. Secure the screw with one of the star nuts. Repeat this procedure with the lower slot in the lateral trunk support. It is important to tighten the first screw only slightly. This makes it easier to tighten the second screw. (fig. 3)

Bring the lateral trunk supports into the preferred position and then tighten both screws. Put the loose end of the lateral trunk support padding under the middle part of the back padding. (fig 4)

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