# Swifty

DE

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GB

## **Seat minimizer**

Assembly instructions – Accessory

FR

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Notice de montage – Accessoires

ES

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The seat minimizer comprises 4 pieces: Head pillow, lateral trunk support, seat cushion, and footrest pad. First, the lateral trunk support is attached to the seat.

**Trunk support:** Position the trunk support on the backrest. The bottom part of the pad must rest on the seat surface. Now, fasten both belts left and right at the cam buckles on bottom rear of the backrest. (Fig. 1a/1b)

**Head pillow:** Guide the h-belt webbing through the loops of the head pillow threading from the top to bottom. Position the head pillow and hook and loop-fasten it to the top pad of the lateral trunk support. (Fig. 2)

**Seat cushion:** Remove the protective strap from the hook and loop (hooks) on the bottom side of the cushion. Place the seat cushion on the seat and hook and loop-fasten it to the bottom of the lateral trunk support. (Fig. 3) Make sure the

hook and loop strap (hooks) does not rest on the interlaced yarns of the Swifty cushion to avoid damages to the cushion.

**Footrest pad:** Place the footrest pad on the footrest. Hook and loop-fasten both webbing straps underneath the footrest plate. (Fig. 4)

**NOTE:** A combined use of Swifty accessories, like lateral trunk supports, soft pelvic harness, seat waistcoat, pelvic belt, or foot straps is not possible or to a limited extent only.

**CAUTION:** Remove the seat minimizer PRIOR TO folding the buggy.

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Thomas Hilfen für Körperbehinderte GmbH & Co. Medico KG · Walkmühlenstr. 1 · D - 27432 Bremervörde · Germany