Swifty 2

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Seat minimizer

Assembly instructions – Accessory

1

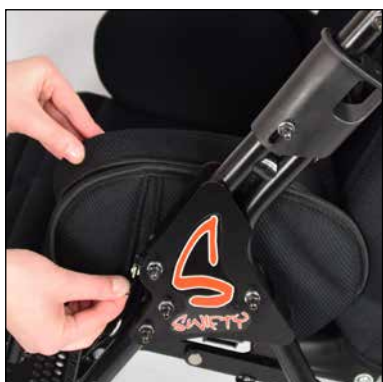
2

3

4

5

6

7

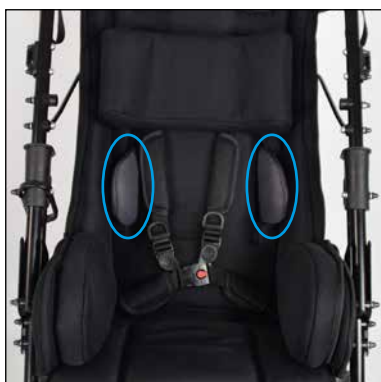
8

9

10

11

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The seat minimizer consists of 4 parts: seat wedge, head pillow and two seat side guide pads. It is started with the fixing of the seat wedge.

Seat wedge:

For positioning of the seat wedge, open the zipper of the back upholstery and remove the upholstery from the back and seat plate. (fig. 1) Put the seat wedge with the broader end on the seat, so that this is leaning directly against the back plate. Now stick the hook and loop strip of the seat wedge onto the seat plate and secure the seat wedge with the hook and loop. (fig. 2) Afterwards pull the upholstery over seat and backrest and close the zipper.

Head pillow:

First, guide the shoulder belts through the buckle of the head pillow from below and fasten the belts on the rear of the back plate. (fig. 3+4) Then the head pillow can be positioned in height with the help of the buckles.

Seat side guide pads:

Open the zipper of the seat side guide pads of the Swifty 2 and remove them from the side plates. Afterwards, pull the seat side guide pads of the seat minimizer over the side plates and close the zippers. Pay attention that the thick upholstery point to the inside of the buggy. (fig. 5)

ATTENTION: When folding the Swifty 2 rehab stroller, flip the upholstery of the seat side guide pads up, so that the upholstery can slide between push handle and back plate. (fig. 6-8)

TIP: For better positioning of the child, the lateral trunk support can be used in addition. Simply attach the pads on the back plate and laid the back upholstery over the pads. (fig. 9+10) Alternatively, the vertical slits in the back upholstery can be cut open to guide the lateral trunk supports through the slits. (fig. 11)

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Thomas Hilfen für Körperbehinderte GmbH & Co. Medico KG · Walkmühlenstr. 1 · D - 27432 Bremervörde · Germany