# tRide · Q Chassis

## **Adapter**

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## **Adapter**

Assembly instructions – Accessory

## **Adattatore**

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### Attaching the adapter

Lock the rear wheel brakes of the Q chassis and align the adapter in such a way that the fixed U-shaped hooks point towards the back. (Fig. 1) Place the adapter on the Q chassis, ensuring that the U-shaped hooks at the back are positioned **between** the spacers on the seat support. Then press the adapter downwards at the front. The adapter must be heard to engage audibly. (Fig. 2)

### Adjusting the tRide seat unit

Pull out the right and left locking pins on the adapter and lock them by turning them through a quarter turn. (Fig. 3) Press and hold down the release mechanism on the rod of the headrest of the tRide seat unit. (Fig. 4) Fit the tRide seat unit with the lateral pins inserted into their recesses in the adapter. Bring the seat unit into the upright position until you hear it audibly engage. (Fig. 5) Allow the lateral locking pins to click into place and then check whether the seat unit is securely engaged by pulling it upwards.

### Removing the tRide seat unit

First pull out the locking pins and lock them in position. Then lift the seat unit upwards to remove it, while pressing the release mechanism (on the rod of the headrest). To remove the adapter, press the release bar forward and lift out the adapter. (Fig. 6)

### Safety instructions

- **Caution:** Watch you do not trap your fingers when the adapter engages.
- Before use, ensure that all the locking pins are engaged. (Fig. 7+8)
- Lock the braked rear wheels of the Q chassis to attach or remove the adapters and the seat unit.
- Fit the adapter and the seat unit the right way round on the Q chassis. (Fig. 9)
- In addition, observe the safety and operating instructions supplied with the tRide Rehab Stroller and the Q chassis.

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Thomas Hilfen für Körperbehinderte GmbH & Co. Medico KG · Walkmühlenstr. 1 · D - 27432 Bremervörde · Germany