# ThevoFlex

Ⓛ	DE	Bedienerhandbuch – ThevoFlex.....	3
Ⓛ	GB	User manual – ThevoFlex.....	6
Ⓛ	FR	Manuel d’utilisation – ThevoFlex.....	9
Ⓛ	ES	Manual de usuario – ThevoFlex.....	12
Ⓛ	IT	Manuale d’istruzioni – ThevoFlex.....	15
Ⓛ	NL	Gebruiksaanwijzing – ThevoFlex.....	18
Ⓛ	NO	Bruksanvisning – ThevoFlex.....	21
Ⓛ	SE	Bruksanvisning – ThevoFlex.....	24
Ⓛ	DK	Brugsanvisning – ThevoFlex.....	27
Ⓛ	FI	Käyttäjän käsikirja – ThevoFlex.....	30

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## Dear User,

You have just acquired a product - the ThevoFlex mobility bed - that meets the highest quality standards. ThevoFlex features the latest materials for an extended service life that's guaranteed.

Please read through this manual carefully before using ThevoFlex. It contains lots of useful information and tips.

If you have any further questions or suggestions concerning ThevoFlex give us a call.

Further information is also available from **[www.thevo.info](http://www.thevo.info)**

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## Contents

## Page

Technical data .....	7
Scope of delivery.....	7
Usage.....	7
Disinfection methods and cleaning information .....	7
Application risks and contraindications.....	7
Warranty conditions.....	8

## Technical data

ThevoFlex external dimensions	90 x 200 cm (35.4 x 78.7 ")	100 x 200 cm (39.4 x 78.7 ")	Height
ThevoFlex 80	16 - 22 kg / 35 - 49 lbs*		19.0 cm (7.5 ")
ThevoFlex 140			

**Other dimensions available on request.**

## Scope of delivery

- ThevoFlex with jersey or incontinence cover
- User manual

## Usage

The product is compatible for use with every type of bed. Lay ThevoFlex with the red side facing upwards on the existing bed slats or bed base.

**Important hint:** ThevoFlex can also be used without a pre-existing slatted frame. In this case the supporting surfaces along the sides of the bed base must be at least 13 cm / 5.1" wide.

**Attention:** Beds with side-rails may require additional side-rail height extenders.

## Disinfection methods and cleaning information

### Mattress

The washable covers are easy to remove thanks to an integrated zip.

### Jersey cover (fabric cover)

- washable at 65 °C (149 °F) • suitable for tumble dryers

### Incontinence cover (rubberised cover)

- washable at 65 °C (149 °F) • suitable for tumble dryers • to be cleaned by wiping disinfection

### Disinfection

We recommend two disinfection methods accepted by the Robert Koch Institute:

- the vacuum/steam/vacuum method up to a maximum temperature of 110 °C (230 °F)
- the steam flow method up to a temperature of 110 °C (230 °F)

**Important hint:** A thermal treatment over 110 °C (230 °F) causes a premature ageing and deformation of the plastic parts and should therefore be avoided. Please make sure when using chemical disinfectants that they are suitable for plastic.

## Application risks and contraindications

ThevoFlex was designed to meet the special needs of wheelchair drivers and combines a pressure relieving mattress with a firm base to facilitate transfers.

ThevoFlex offers a firm surface for lying. Skin condition should be checked regularly for persons with, or at risk for skin breakdown. Selecting the correct mattress density is extremely important, as well.

The use of ThevoFlex does not require any special instructions. After practical use and conducting several tests, we are unaware of any problems associated with the use of this product.

When used properly, there are no known risks.

**Important hint:** For the correct supply it is important to ensure that the mattress is selected according to the body weight of the user in question.

- **ThevoFlex 80** up to a body weight of 80 kg (176.4 lbs)
- **ThevoFlex 140** from 80 kg (176.4 lbs) up to a body weight of 140 kg (308.6 lbs)

ThevoFlex 140 may also be used in the border zone of patients weighing 80 kg/176.4 lbs (+/- 2 kg / +/- 4.4 lbs).

**Caution:** ThevoFlex is not suitable for the prevention and treatment of decubitus ulcers. This requires the use of either ThevoautoActiv or ThevoActiv. We would be pleased to advise you

## Warranty conditions

The company Thomashilfen takes full responsibility for the quality of the product supplied. The highest production standards are used in the selection and processing of materials. All ThevoFlex mattresses are subjected to a thorough final inspection. Thomashilfen offers you a warranty of 24 months. Please retain all relevant ThevoFlex receipts. The warranty covers all claims arising from manufacturing or material defects that affect product function. Damage arising from inappropriate use and natural wear and tear is excluded.

**Important hint:** ThevoFlex is autoclavable and may be re-used. ThevoFlex should be cleaned and disinfected using one of the cleaning methods stated on page 7 with every change of user. Servicing by the manufacturer is not necessary.

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