ThevoRelief

DE	Bedienerhandbuch – ThevoRelief	3
GB	User manual – ThevoRelief.....	6
FR	Manuel d’utilisation – ThevoRelief	9
ES	Manual de usuario – ThevoRelief	12
IT	Manuale d’istruzioni – ThevoRelief	15
NL	Gebruiksaanwijzing – ThevoRelief.....	18
NO	Bruksanvisning – ThevoRelief	21
SE	Bruksanvisning – ThevoRelief	24
DK	Brugsanvisning – ThevoRelief	27
FI	Käyttäjän käsikirja – ThevoRelief.....	30

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Dear User,

You have just acquired a product - the ThevoRelief mobility bed - that meets the highest quality standards. ThevoRelief features the latest materials for an extended service life that's guaranteed.

Please read through this manual carefully before using ThevoRelief. It contains lots of useful information and tips. If you have any further questions or suggestions concerning ThevoRelief give us a call.

Further information is also available from **www.thevo.info**

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Contents

Page

Technical data	7
Scope of delivery.....	7
Usage.....	7
Disinfection methods and cleaning information	7
Application risks and contraindications.....	7
Warranty conditions.....	8

Technical data

ThevoRelief external dimensions	90 x 200 cm (35.4 x 78.7 ")	100 x 200 cm (39.4 x 78.7 ")	Height
ThevoRelief 50	20 - 25 kg / 44 - 55 lbs*		19.0 cm (7.5 ")
Thevo Relief 100			
ThevoRelief 135			

Other dimensions available on request.

Scope of delivery

- ThevoRelief with jersey or incontinence cover
- User manual

Usage

The product is compatible for use with every type of bed. Lay ThevoRelief with the red side facing upwards on the existing bed slats or bed base.

Important hint: ThevoRelief can also be used without a pre-existing slatted frame. In this case the supporting surfaces along the sides of the bed base must be at least 13 cm (5.1") wide.

Attention: Beds with side-rails may require additional side-rail height extenders.

Disinfection methods and cleaning information

Mattress

The washable covers are easy to remove thanks to an integrated zip.

Jersey cover (fabric cover)

- washable at 65 °C (149 °F) • suitable for tumble dryers

Incontinence cover (rubberised cover)

- washable at 65 °C (149 °F) • suitable for tumble dryers • to be cleaned by wiping disinfection

Disinfection

We recommend two disinfection methods accepted by the Robert Koch Institute:

- the vacuum/steam/vacuum method up to a maximum temperature of 110 °C (230 °F)
- the steam flow method up to a temperature of 110 °C (230 °F)

Important hint: A thermal treatment over 110 °C (230 °F) causes a premature ageing and deformation of the plastic parts and should therefore be avoided. Please make sure when using chemical disinfectants that they are suitable for plastic.

Application risks and contraindications

ThevoRelief was designed to relieve pain when lying down and improve the quality of sleep of persons suffering from arthrosis, osteoporosis, or cancer.

The use of ThevoRelief does not require any special instructions. After practical use and conducting several tests, we are unaware of any problems associated with the use of this product.

When used properly, there are no known risks.

Important hint: For the correct supply it is important to ensure that the mattress is selected according to the body weight of the user in question.

- **ThevoRelief 50** up to a body weight of 50 kg (110.2 lbs)
- **ThevoRelief 100** from 50 kg (110.2 lbs) up to a body weight of 100 kg (220.4 lbs)
- **ThevoRelief 135** from 100 kg (220.4 lbs) up to a body weight of 135 kg (297.6 lbs)

ThevoRelief 100 may also be used in the border zone of patients weighing 50 kg/110.2 lbs (+/- 2 kg / +/- 4.4 lbs).

ThevoRelief 135 may also be used in the border zone of patients weighing 100 kg/220.4 lbs (+/- 2 kg / +/- 4.4 lbs).

Caution: ThevoRelief is not suitable for the prevention and treatment of decubitus ulcers. This requires the use of either ThevoautoActiv or ThevoActiv. We would be pleased to advise you

Warranty conditions

The company Thomashilfen takes full responsibility for the quality of the product supplied. The highest production standards are used in the selection and processing of materials. All ThevoRelief mattresses are subjected to a thorough final inspection. Thomashilfen offers you a warranty of 24 months. Please retain all relevant ThevoRelief receipts. The warranty covers all claims arising from manufacturing or material defects that affect product function. Damage arising from inappropriate use and natural wear and tear is excluded.

Important hint: ThevoRelief is autoclavable and may be re-used. ThevoRelief should be cleaned and disinfected using one of the cleaning methods stated on page 7 with every change of user. Servicing by the manufacturer is not necessary.

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